

Zeus SP & East SP

Medical Gas Terminal Units



Zeus SP terminal unit

Combining ease and speed of installation

The Pneumatech Medical Gas Solutions (PMGS) Zeus SP and East SP medical gas terminal units are designed to accept medical gas probes to BS 5682:1998. They provide a safe means of supplying medical equipment with medical gases from the central gas supply system.



The construction of PMGS' Zeus SP is of machined brass and injection moulded polymer with stainless steel Teflon[™] coated rolling pins. Each of the gas specific components have the gas service placed onto it to ensure safety and compliance with the standard. The wall mounted terminal units incorporate an anti-swivel pin to prevent rotation of installed equipment.

PMGS medical gas terminal units are enclosed in a white ABS decorative mounting box and the fascia have rounded corners to avoid the possibility of injury. The box can be supplied with a flush mounting bezel as a plaster finish.

Design

All PMGS' Zeus SP terminal units individually undergo a pressure and leak test to ensure the safety of our products. Integral check valves allow removal of the housing and socket assemblies for maintenance without closing down the entire pipeline. Removal of second fix can easily be achieved without dismantling other parts.

A combination of Teflon[™] and stainless steel rolling pins gives a smooth action for the insertion and removal of the probe.

Safety

To ensure safety, PMGS makes use of clear gas indication on the Zeus SP terminal units, which significantly reduces the risk of connecting a second fix from one service to the first fix of another service, either during installation or service. A positive action of the rolling pin latch mechanism holds the probe securely.

Gases and Services

The gases and services Pneumatech MGS provides:

- Oxygen
- Nitrous Oxide
- 50% Oxygen / 50% Nitrous Oxide
- Medical Air (400 kPa)
- Surgical Air (400 kPa)
- Medical Vacuum
- Nitrogen
- Carbon Dioxide

Component list

- First fix with testing plug
- Second fix with valve
- Installation kit

East SP terminal unit

The East SP medical gas terminal unit incorporates a sheerplane that ensures a fail-safe condition after accidental damage or bed jacking (causing no damage to first fix and enabling easy replacement without isolation). Construction of the terminal unit is of machined brass and die-cast chrome collar with stainless steel rolling pins. Each of the gas specific components has the gas service engraved onto it, to ensure safety and compliance with the standard.

As with the Zeus SP, the East SP terminal unit is enclosed in a white ABS decorative mounting box with an ABS fascia.



Design

Pneumatech MGS' East SP terminal units consist out of an all metal construction. As the Zeus SP range, these terminal units undergo strict pressure and leak tests to ensure the safety of our products. A full metal to metal seal on the maintenance check valve ensures no degradation over time. Stainless steel rolling pins give a smooth action for the insertion and removal of the probe.

Safety

The East SP medical gas terminal units of PMGS contain clear gas indication, reducing the risk of connecting a socket assembly of one gas to terminal block to another, either during installation or maintenance. An anti rotational locking bar and the gas indexing pin are cast into the East SP socket assembly and can't come loose or be removed.

8102341065 ed.1 - @ 2015, Pneumatech Medical Gas Solutions. All rights reserved. Designs and specifications are subject to change without notice or obligation.

Key Points

Factory tested and certified.

High quality components.

Integral check valve.

Conversion terminal units available.

MRI compatible.

All metal construction (only for East SP)

Compliance

HTM02-01, HTM2022, C11 and all related BS EN ISO standards.

Easy installation

Available as a complete kit, or as separate first and second fix components so that supply can be matched to the progress of site work at larger installations.

First fix incorporates a copper stub for ease of installation using inert gas jointing procedures.

Mounting Options

Wall mounted, flush or surface fitting

Bedhead trunking

Pendant or boom, NIST connection

Formerly known as Penlon Medical Gas Solutions





Service and maintenance

highest quality.

similar local service.

published service schedule.

Throughout the UK our network of service engineers

provides full Service and Maintenance support of the

Outside the UK our fully trained distributors provide a

Three year warranty from date of manufacture,

items to be replaced by the customer as part of the

excluding components designated as service

www.p-mgs.com